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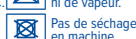
LOT Katso pakkausta.

CORRECTIVE ORTHOSIS FOR HALLUX VALGUS

INDICATIONS: Wearing the EPITACT® soft orthosis daily helps correct the hallux valgus deformity and relieves associated pain.

CONTRAINDICATIONS: Great toe that does not move. Do not use on a skin lesion. Persons suffering from arteritis, diabetes, neuropathy of the lower limbs or varicose ulcers: never use the product on a wound, monitor the feet during use and ask your doctor or chiropodist for advice.

PRECAUTIONS FOR USE

- Do not use this product overnight.
- The orthosis can be used on either the right or the left foot.
- Wash before first use.

SET-UP :

- 1 Put the brace on your foot.
- 2 Place the cap over the big toe and align the target with the bunion.
- 3 If the toe is still misaligned, gradually pull the strap until the toe is straightened.

- During the first days of use, you may apply a light correction and increase it gradually as you get used to the device.
- The device should be worn daily for maximum efficiency.

CARE: Wash after use at 30°C in the washing bag supplied or wash by hand (without rubbing). Discard and replace the device as soon as cracks in the gel or fabric or seam damages occur. Leave to air-dry. Do not iron.

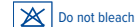
COMPOSITION:

51% polyamide/elastane fabric, 49% silicone.

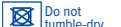
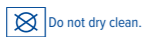
In case of pain or any other unusual reaction, contact your doctor. The silicone tendon in contact with the skin may cause mechanical irritations, such as blisters (< 0.1%). Any major incidents involving the device must be reported to the manufacturer and the relevant health authority. Millet Innovation cannot be held responsible for failure to use the product in accordance with the manufacturer's recommendations. French is the reference language. CE initial marking: 2011.

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